FFP3 DISPOSABLE PROTECTIVE FACE MASKS

FFP3 protective masks are EM & medical grade and offer a filter penetration limit of at least 99% of airborne particles (95L/min). Designed to have a good fit and complete seal for all face types and sizes and is disposable and for single use. For civilian use only we also have KN95 Face Masks available and in stock.

Standards applied:

- FFP3 (99%+ partial filter penetration limit)
- EU MDD Directive 93/42/EEC on Medical Devices as a Class 1 Device
- EU Regulation (EU) 2017/745 on medical devices as a Class 1 Medical Device, with original implementation date May 2020, without the requirement for Regulation 2020/561 postponing implementation.
- EU PPE Regulation 2016/425. The product is marked as a Class I PPE device and meets all requirements for a Class III PPE device for filtration to FFP3 level with particles down to 100nm in diameter.
- EN1822-4 for biological protection
- EN:149:2001-A1:2009 Dust and biological protection masks
- EN 1073-2 Barrier to radioactive particulates

Features:

- CE marked
- Non-valved
- Elasticated straps
- Comfortable to wear
- Soft fabric construction
- Disposable
- · Metal nose strip for a good seal
- · Each mask packaged separately

Fitting instructions:

Before handling the product you must ensure your hands are clean, do not use if damaged.

- 1 Unfold FFP3 mask to form a cup shape
- 2 Hold mask with one hand and move straps to the outside of the mask
- 3 Place mask over face and place straps overhead, making sure the straps do not cross
- 4 Conform nose clip to form a tight seal to the nose using both hands
- 5 Inhale sharply, this will check if the air leaks
- 6 If air leaks around the nose adjust the nose clip
- 7 If air leaks around the face adjust the straps
- 8 If air leaks cannot be prevented then do not use the mask

NORTHERN DIVER

+44 (0) 1257 25 44 44 | info@ndiver.com For more visit www.ndiver-rescue.com

DESCRIPTION	STOCK CODE
PPE	MASK-FFP3



